

Chapter 3 Troubleshooting

3.2.12 06550 : Optical Unit/Eraser Unit Errors

- Error codes S column: Errors at the read operation/calibration.
- Error codes E column: Errors during erase operation.
- Error codes I column: Errors during initialization.

Error code			Description of the error	Response (References to replacement methods are listed at the end of the table.)
S	E	I		
06550	-	-	Polygon rotation error Rotation of the polygon was not stable when the push plate unit moved to the read position.	Replace the optical unit.

References for response method

Procedure for replacing optical unit	"5.2.5 Removing/Installing the Optical Unit"
--------------------------------------	--

3.2.13 06601 - 06603: Fan Error

Error code	Description of the error	Response (References to replacement methods are listed at the end of the table.)
06601	Eraser cooling fan blowout Fan blowout was detected when the cooling fan tried to turn.	Confirm following and replace the faulty parts. <ul style="list-style-type: none"> • Eraser cooling fan • Fan wire: JJ19 - MCN1 (MDU3) • MDU3
06602	Digital power supply cooling fan blowout Fan blowout was detected when the cooling fan tried to turn.	Confirm following and replace the faulty parts. <ul style="list-style-type: none"> • Digital power supply cooling fan • Fan wire: JJ49 - JP11/JJ11 - MCN9 (MDU3) • MDU3
06603	MDU cooling fan blowout Fan blowout was detected when the cooling fan tried to turn.	Confirm following and replace the faulty parts. <ul style="list-style-type: none"> • MDU cooling fan • MDU3

References for response method

Position of eraser cooling fan	"2.5.1 Insertion Unit"
Position of digital power supply cooling fan	"2.5.7 Exterior"
Position of MDU3	"2.5.8 Framework"
Procedure for replacing eraser cooling fan	"5.3.3 Replacing the Eraser Cooling Fan"
Procedure for replacing digital power supply cooling fan	"5.9.15 Replacing the Digital Power Supply Cooling Fan"
Procedure for replacing MDU3	"5.10.2 Replacing the MDU3"