	Release Notes	Ausgabe	Seite
	MT-200	10.08.2004	1/3

☐ **Development** (Confidential)

EM

☐ **internal** (Confidential)

VS

☒ **external (CD)**

VM UB/10.08.04

Unit: -

Software: MT-200

Version

: 2.04

Date: 10.08.2004

Language: GER, ENG

Written by: SH / UB

<u>Actual Device Software</u>	
MT-200 Version	: 2.04

<u>Additional Informations</u>	
File Name	: MT200.exe
Built Date	: 09.08.2004 (Time 15:29:02)

<u>Reference Documents</u>
MT200_V2_00_NewsLetter.doc


NOTE:

Adjustment of the Schiller CD !

WinNT The USB drivers for WinNT should only be used for the AT-110 and PT-160. This should be checked with the support when the MT-200 is installed on a WinNT PC (not recommended).

Audio Data : The voice recording functions require a PC with a sound card!

SEMA 200 : The MT-200 version 2.0 is compatible with SEMA-200 versions 2.15 and higher.

	Release Notes	Ausgabe	Seite
	MT-200	02.07.2004	2/3

General

1. Languages

This release version is available in the following languages :


- German
- English

The MT-200 must be restarted after the language has been changed.

1.1 Europe / America

The colour codes for electrode cables are different for Europe and America.

The valid standard for Europe is EN 60601-2-47; the standard for America is IEC 60601-2-47. The applicable colour code is defined upon the device's installation. See the following picture:

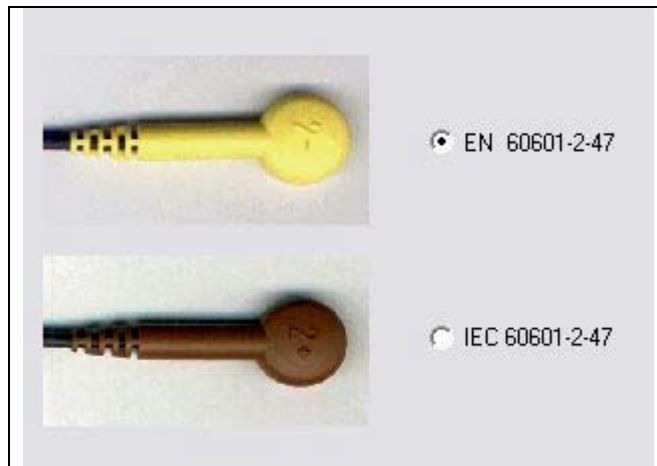


2. Operating Systems

The MT-200 can be installed on the following operating systems:

Win98 SE : - with USB 1.0
Win2000 : - with USB 1.0/2.0
WinXP : - with USB 1.0/2.0

Special WinNT for PT-160/AT-110

- To enable access to the MT-101/MT-100, install MT-200 WinNT USB Driver_9_10_01.exe.

	Release Notes	Ausgabe	Seite
	MT-200	02.07.2004	3/3

Remarks

3. Pacemaker

The MT-101 hardware can only detect a pacemaker on channel I. The MT-200 indicates markers on all displayed leads.

4. Languages

All modifications are displayed in English in the different language versions (except the German version). As soon as all modifications have been translated into the relevant languages, there will be a new language update for the MT-200.

New Functions

UNC path for SD-card possible.