


Update software of BC2800/BC3000/BC1800(new version)

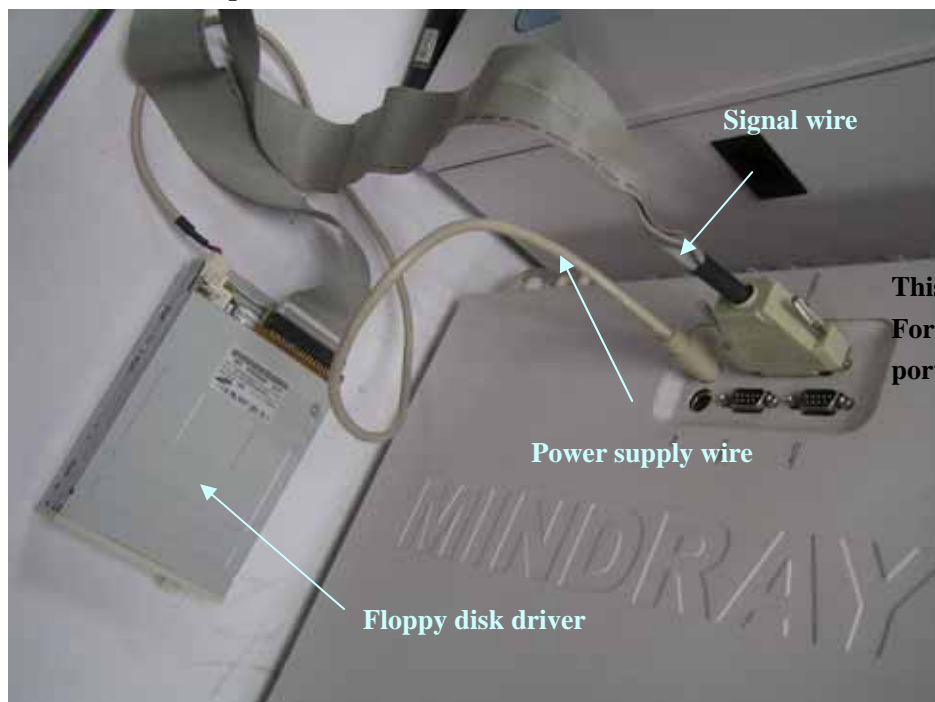

Tools needed:

Floppy disk driver, floppy disk, cables connect the instrument and floppy disk driver (data cable and power supply cable)

Procedure:

1. Copy all the unzipped upgrade software to a floppy disk which should be formatted first. (If the volume of one floppy disk is not enough, please open the file of UPDATE.CFG with txt and ordinal copy the unzipped upgrade software to several floppy disks according to the sequence of files listed in UPDATE.CFG. Please note UPDATE.CFG should be in the first floppy disk.)
2. Connect a keyboard and a floppy disk driver to the unit, turn on the hematology analyzer and wait until the system accesses the “Count” screen.
3. Press “Menu”, move the cursor onto “Setup/Password” and press [Enter] key to access “Password” screen. Enter “3210” and then return to “Count” screen.
4. Press “up” button, the HEMATOLOGY ANALYZER system software will read the floppy disk and show following information.
“Insert disk of config, then press [ENTER] key.”
5. Insert the first floppy disk and press [ENTER], then system will show:
“Reading config file ...”
6. System will read data from floppy disk, if OK, it will show.
“Insert disk of data, then press [ENTER] key.”
8. If one floppy disk is not enough, it will show.
“Insert disk of following data, then press [ENTER] key. ”
9. Press [ENTER], then system will show:
“Writing files to disk or flash ...”
10. System will copy files to disk on module.
“Update is succeeded, wait for restarting ...”
11. Then system will re-start, and upgrade finish.